

# HÖRLUCHS HEARING GMBH & CO. KG

## HEADQUARTERS

Bergseestraße 10, 91217 Hersbruck, Germany

## DELIVERY ADDRESS GOODS & ORDERS

Friedrich-Köchert-Str. 4, 91217 Hersbruck, Germany

TEL: +49 9151 90886-0

FAX: +49 9151 90886-27

EMAIL: [info@hoerluchs.com](mailto:info@hoerluchs.com)

WEB: [www.hoerluchs.com](http://www.hoerluchs.com)

# HÖRLUCHS®



MADE IN GERMANY

## USER INFORMATION FOR THE HÖRLUCHS® PRODUCTS SOWEI SLEEP (IIC), SOWEI AQUA & SOWEI SURF

### CORRECT INSERTION OF THE EARMOULD

Your personal comfort protection **Hörluchs® SOWEI SLEEP, SOWEI SLEEP IIC, SOWEI AQUA** or **SOWEI SURF** is marked with a side mark. The right earmould with „R“, the left with „L“. **Please make sure that the product is always inserted in the correct ear.**

Always grip your Hörluchs® product between your thumb and index finger and start with the right earmould. Place the ear canal pin of the product into the ear canal. With a slight forward twist, insert it into the pinna. In some cases, insertion is facilitated by gently pulling on the earlobe. Make sure that the earmould fits into the upper ear fold (for concha products).

### PLEASE NOTE

Regularly check that the product is not damaged and that it is always fully functional. Avoid deliberately overstretching the silicone material. We recommend a regular check by your hearing care professional.

### ATTENTION

**The above products have not been tested and approved as hearing protection! Use as PPE (personal protective equipment) is therefore not permitted.**

To protect your ear effectively against dangerous and harmful noise, please always use hearing protection with type approval!

### INTENDED USE

Your product is intended to increase comfort in the event of disturbing noises or to protect against the penetration of splash water (when correctly fitted and attached). **Any other use is not intended and therefore not permitted!**

### STORAGE & CLEANING

Always store your personal Hörluchs® product in the supplied case when not in use.

**Clean your product regularly.** We recommend our auriFIX cleaning spray with brush. Water and mild soap can also be used. When completely dry, please store your earmould in the protective case provided. This product may suffer from certain chemical substances. Further information can be obtained from the manufacturer.

If the earmould is stored in a dry, clean place and the instructions for storage and care are followed, it will last for five years from the date of manufacture. The date of manufacture can be found on the back of the case. **For transport and storage, the recommended temperature range is between 0°-35° C. Storage at more than 55° C is not permitted!**

EN

Version: 20211206a